



Female Catheter Model

LF01151U

INSTRUCTION CARD



5-Year Warranty

Actual product may vary slightly from photo. Nasco reserves the right to change product color, materials, supplies, or function as needed.

USING THE MODEL

- Use this model to practice sterile technique and proper cleansing procedures before, during, and after insertion.
- The model will accept any catheter size 16 or smaller.
- Be sure the balloon on the catheter is fully deflated before insertion or removal. Always lubricate the catheter thoroughly before each insertion.
- Spread the labia, locate the urethral meatus between the clitoris and vagina, and gently insert the catheter until it is positioned within the bladder (about 7.5 cm). Proper insertion in a live patient is evidenced by a flow of urine through the tube.

CARING FOR THE MODEL


- The model may be washed in mild soap and water if it becomes soiled.
- Always store it in the plastic bag when not in use.
- Printed material or ballpoint pens will transfer indelible stains to the model. Certain finishes may also be damaged from prolonged contact with the model.

AVAILABLE SUPPLIES

LF01127U 16 FR Foley Catheter

LF01128U 16 FR Foley Catheter, 10-pack

LF00985U Lubricant Kit, 2 oz. bottles, 6-pack

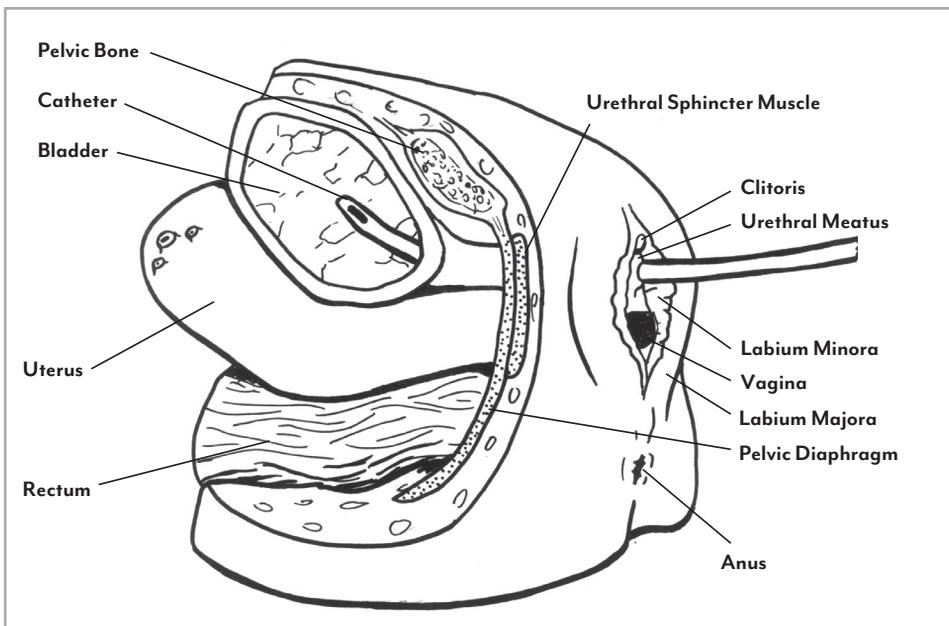

ONLINE:
EMAIL:
PHONE:
ADDRESS:

NascoHealthcare.com
lifeform@enasco.com
1.800.431.4310
901 Janesville Ave.
Fort Atkinson, WI 53538